# Glucosefine<sup>®</sup> Pro

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## Glucosefine<sup>®</sup> Pro stechhilfe

TaiDoc Technology Corporation  
B1-7F, No. 127, Wugong 2nd Rd., Wugu Dist.,  
24888 New Taipei City, Taiwan  
www.taidoc.com

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0483 PZN 11537140  
4226009  
318-5010200-XXX  
GFP-LDM-MTR Rev. 2021-04-14

# Glucosefine<sup>®</sup> Pro


## Instruction manual for lancing device and sterile lancet

### Intended use:


The Glucosefine<sup>®</sup> Pro lancing device allows to collect capillary blood samples with sterile lancets for glucose monitoring or for other tests from the finger tip or alternate sites easily and almost painless.

### Product overview and features:

#### Lancing device

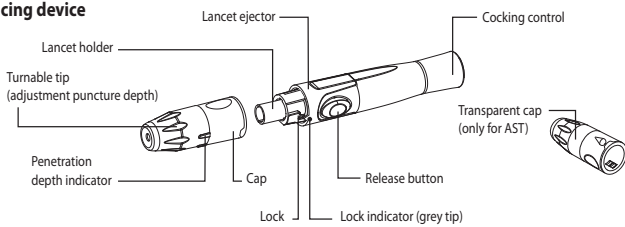
#### The lancing device allows to set 6 different penetration depths.
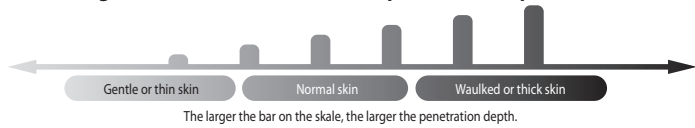
#### Sterile lancet

##### BEFORE use:

Turn protective cap (1) and remove (2)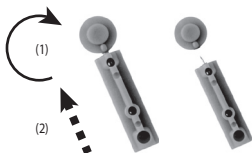


AFTER use: Put on protection cap with the flat side and discard.

### Important notes:


To reduce the infection risk:

- Never share the lancet or the lancing device.
- Always use a new, sterile lancet. Lancets are for single use only.
- Avoid getting hand lotion, oils, dirt or debris in or on the lancets and the lancing device.
- The used lancet may potentially be biohazardous. Discard it accordingly to your local regulations.
- Do not use in case of a damaged or missing protective cap.
- Do not keep the lancet in the lancing device.
- Do not use the lancets after expiration date. Sterility will maintain for 5 years. Do not sterilize again. Glucosefine<sup>®</sup> Pro lancets are only intended for use with the Glucosefine<sup>®</sup> or Glucosefine<sup>®</sup> Pro lancing device.

### Setting up the lancing device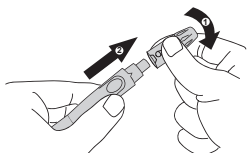


1. Unscrew the cap of the lancing device by turning it counter-clockwise while holding the base firmly.

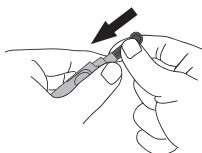
2. While holding the lancet carrier, insert a new sterile lancet and push it down firmly until it is fully secured.


3. Twist the cap of the lancet off.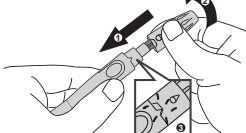

4. Put the cap of the lancing device back on and turn it clockwise until it snaps audibly.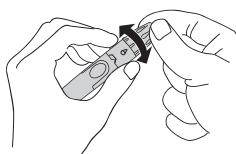

5. Turn the tip until the mark (white drop) points to the requested penetration depth. You can adjust 6 penetration depths.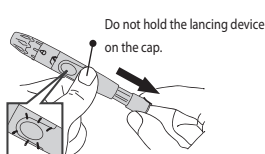


6. Hold the tip firmly in one hand then pull out the cocking control with the other hand until „click“-sound resonates. The release button turns red as soon as the lancing device is cocked.  
**► Should there be no "click"-sound, you may have cocked the lancing device with inserting the lancet.**

### Caution:

Before collecting the blood sample:


- Wash your hands dry completely before testing.
- Choose the puncture side.
- For a well blood circulation, please massage the skin on the puncture side for about 20 seconds to encourage blood flow.

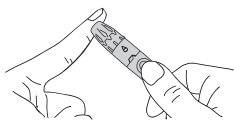7. Puncture  
Press the lancing device firmly against your finger tip and press the release button.

8. Remove the device from the puncture site. After penetration, remove the first drop of blood with a clean cotton swab. Gently squeeze the punctured area to obtain another drop of blood. Be careful NOT to smear the blood sample.

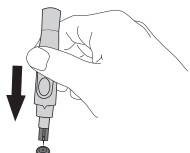9. Remove the lancet by twisting the cap off and safely dispose of the used lancet by pushing the exposed tip into the desk.


10. Slide the ejector forward to remove the used lancet. Dispose of the lancet according to your local regulations.  
**► Please use the transparent cap for AST.**

### Alternative Site Testing (AST)

Testing on sites other than the fingertip is referred to as Alternative site testing (AST). Please read the Alternative Site Testing (AST) section in your operation manual before you perform AST.

Substitute the lancing device cap with the clear cap for alternative site testing. Pull the cocking control back until it clicks.

When lancing the forearm, upper arm, hand, thigh, or calf, avoid lancing the areas with obvious veins in order to prevent excess bleeding.

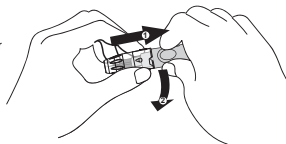### Cleaning

The lancet is disposable and intended for single use only.

To clean the lancing device, use a mild soap and water to wipe clean the lancing device. Do not rinse the device under water. Desinfect the cap by placing it in 70 % alcohol for 10 minutes and then allow it to air dry.

Do not place the device in a dishwasher or use detergents.

### Symbols

#### Symbol Explanation

	Attention - See further instructions for use
	Consult instructions for use!
	Only for single use
	Manufacturer
	Use by
	This product meets the requirements of the Regulation (EU) 2017/745

#### Symbol Explanation

	Sterilized using irradiation
	Do not use if packaging is damaged
	Medical device
	EU Authorized Representative for Medical Devices
	Batch code
	Reference number
	This product fulfills the requirements of Directive 93/42/EC on medical devices

### Product specifications

#### Glucosefine<sup>®</sup> Pro lancing device

Article number	Total length	Max. Ø
42980105	120 mm	18 mm

### Product specifications

#### Glucosefine<sup>®</sup> Pro lancets, 30G

Article number	PZN	Total length	Max. Ø
4226009	11537140	32 mm	6,4 mm

## Glucosefine<sup>®</sup> Pro Lancing device

TaiDoc Technology Corporation  
 B1-7F, No. 127, Wugong 2nd Rd., Wugu Dist.,  
 24888 New Taipei City, Taiwan  
 www.taidoc.com

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MedNet EC-REP GmbH
Metrado GmbH